



S-Mono




S-Mono

OneBody Mini Implant

- Adequate for narrow upper and lower anterior teeth
- One-piece design with high strength for bite force
- RBM surface for enhancing osseointegration

S-Mono Line-up

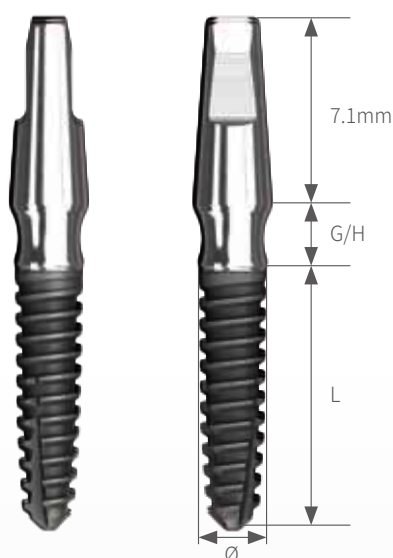

Length (mm)	G/H (mm)	Ø2.5	Ø3.0
8.5	2.5	MB2508SR	MB3008SR
10		MB2510SR	MB3010SR
11.5		MB2511SR	MB3011SR
13		MB2513SR	MB3013SR
8.5	4	MB2508LR	MB3008LR
10		MB2510LR	MB3010LR
11.5		MB2511LR	MB3011LR
13		MB2513LR	MB3013LR

Impression Coping
Used to take impressions

MBICP



Temporary Cap
Used for a temporary prosthesis

MBTCP



Lab Analog
Used for dental laboratory

MBAP



S-Mono Surgical Kit



S-Mono Kit(MBKT)

Drill Extension

Point Drill

Ø1.8 Drill Short

Ø1.8 Drill Long

Ø2.3 Drill Short

Ø2.3 Drill Long

Direction Indicator

Fixture Driver T/W

Fixture Driver H/P

Torque Wrench
(Option)



SDE00



MBPD15S



MBSD18S



MBSD18L



MBSD23S



MBSD23L



MBDI00



TWFD00




HFDP00



TW00

S-Mono Surgical Procedure

- **Drilling Speed** : Recommend 1200~1500rpm
- **Insertion Speed** : Recommend 20~40rpm
- **Insertion Torque** : Recommend under 45Ncm

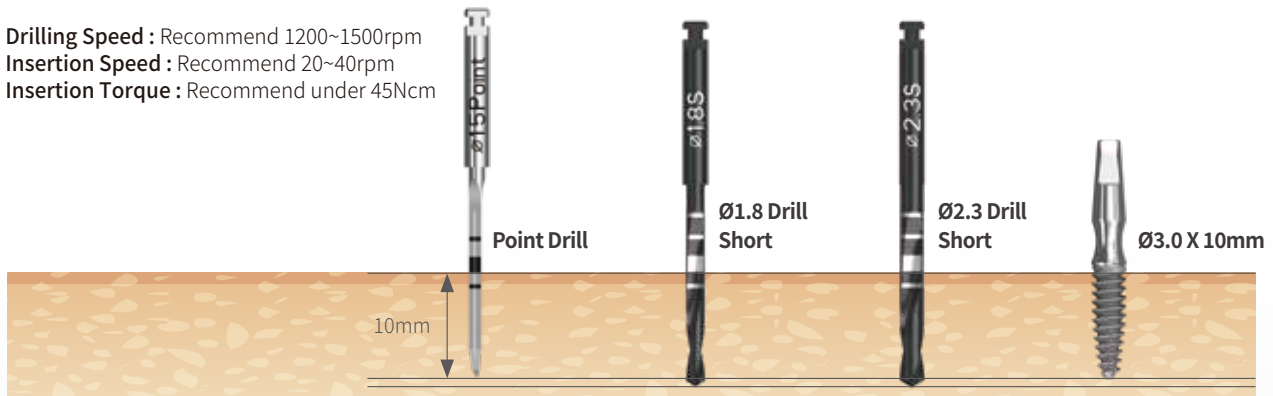

	Bone quality	Point Drill	Ø1.8 Drill	Ø2.3 Drill	Fixture
Ø2.5	Soft	○			
	Medium	○	○		
	Dense	○	○		
Ø3.0	Soft	○	○		
	Medium	○	○	○	
	Dense	○	○	○	

S-Mono

SIS-DENTAL INC

Shinhung.co.,Ltd.

450, Cheongpa-ro, Jung-gu, Seoul, Korea
Tel : 82-2-6366-2175 / Fax : 82-2-6366-2131
E-mail : export@shinhung.co.kr
www.shinhung.com

SIS-DENTAL INC

6131 Orangethorpe Ave Suite 410, Buena Park, CA 90620, USA
Tel : 562-550-0001
E-mail : yohan@SIS-Dental.com
www.sisdentalusa.com